




Finger Stick blood collection instructions

Collection kit supplied by Oncimmune[®]



Complete and sign
EarlyCDT—Liver test requisition

Patient should wash
and warm hands

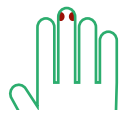

Identify puncture site (red).
Clean thoroughly with alcohol
wipe & air dry

Twist off tab
on lancet



Press firmly against
puncture site until
click is heard

Return with sample for proper disposal


Wipe away first drop of blood.
Remove yellow cap of Microtainer[®] tube



Touch collector end
of tube to drop of blood,
apply intermittent pressure
along finger*

Note: some may need more than 1 finger stick

Fill tube to the 400 mark or above.
Replace cap and label tube
with barcode sticker

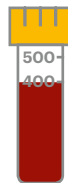

Prepare for shipment:

- Place sample in bubble wrap provided.
- Place sample and lancets in biohazard bag.
- Place bag and requisition in collection kit box.
- Place box in FedEx[®] Clinical Pak with prepaid waybill affixed.


Traditional blood draw instructions

Collection kit supplied by Oncimmune[®]



Complete and sign
EarlyCDT—Liver test requisition


Blood specimen drawn into a
red-top serum tube or serum-
separator tube (SST)

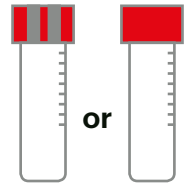

If SST, centrifuge specimen per
manufacturer's recommended procedures.
If sending red-top serum tube, send
specimen without processing

Label tube with 2 unique patient
identifiers and/or barcode label

Note: minimum of 0.5 mL of serum
or 1.0 mL of whole blood

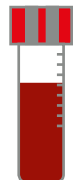

Keep refrigerated
(not frozen) until ready to
ship, unless shipped same day



Prepare for shipment:

- Place sample in bubble wrap provided.
- Place sample in biohazard bag.
- Place bag and requisition in collection kit box.
- Place box in FedEx[®] Clinical Pak with prepaid waybill affixed.

*Do not scrape skin surface to collect blood; blood will freely flow to bottom of tube.